TEKNO

TEKNOmini

ISTRUZIONI

DIRECTIONS

INSTRUCCIONES

TEKNO
TEKNomini
ISTRUZIONI
INSTRUCTIONS
INSTRUCCIONES

ISTRTEKNO

DATA: 10.02.2025 REV.2

PAGINA: 3/32

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Capitolo 1/ Chapter 1/ Capítulo 1	4-6	Capitolo 5/ Chapter 5/ Capítulo 5/	25-26
1.1 Campo di impiego 1.1 Field of use 1.1 Campo de empleo		5.1 Imballo e trasporto 5.1 Packaging and transport 5.1 Embalaje y transporte	
1.2 Generalità 1.2 General 1.2 Datos generales		Allegato A/ Annex A/ Anexo A/	27
1.3 Installazione 1.3 Installation 1.3 Instalación		AVVERTENZE PRECAUTIONS ADVERTENCIAS	
Capitolo 2/ Chapter 2/ Capítulo 2	7-20	Allegato B/ Annex B/ Anexo B/	28
2.1 Simbologia / Key / Simbología		DICHIARAZIONE DI CONFORMITA' DECLARATION OF CONFORMITY DECLARACION DE CONFORMIDAD	
2.2 Montaggio del tavolo 2.2 Assembly of the table 2.2 Montaje de la mesa		Allegato C/ Annex C/ Anexo C/	29-31
Capitolo 3/ Chapter 3/ Capítulo 3/	21-22	TERMINI DI GARANZIA DEI PRODOTTI ERIO S.r.l. WARRANTY TERMS OF ERIO PRODUCTS CONDICIONES DE GARANTÍA DE LOS PRODUCTOS	
3.1 Caratteristiche Tecniche 3.1 Technical features 3.1 Características técnicas		Allegato D / Annex D / Anexo D	32
3.2 Lista dei Pezzi di ricambio 3.2 Spare parts list 3.2 Lista de piezas de repuesto		INFORMAZIONI PER LO SMALTIMENTO DEGLI IMBALLI INFORMATION ON DISPOSAL OF PACKAGING INFORMACIÓN PARA LA ELIMINACIÓN DE EMBALAJES	
3.3 Schema elettrico 3.3 Wiring diagram 3.3 Instalacion eléctrica			
3.4 Manutenzione ordinaria del tavolo 3.4 Ordinary maintenance of the table 3.4 Mantenimiento ordinario de la mesa			
Capitolo 4/ Chapter 4/ Capítulo 4	23-24		
4.1 Sostituzione dei filtri 4.1 Filters replacement 4.1 Sustitución de los filtros			
4.2 Sostituzione del motore di aspirazione 4.2 Replacement of the extractor motor 4.2 Sustitución del motor de aspiración			
4.3 Sostituzione del fusibile 4.3 Changing the fuse 4.3 Sustitución del fusibile			
4.4 Pulizia 4.4 Cleaning 4.4 Limpieza			

TEKNO
TEKNOmini
ISTRUZIONI
INSTRUCTIONS
INSTRUCCIONES

ISTRTEKNO

DATA: 10.02.2025 REV.2

PAGINA: 4/32

1

Capitolo 1 / Chapter 1 / Capítulo 1

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1.1 Field of use

1.1 Campo de empleo

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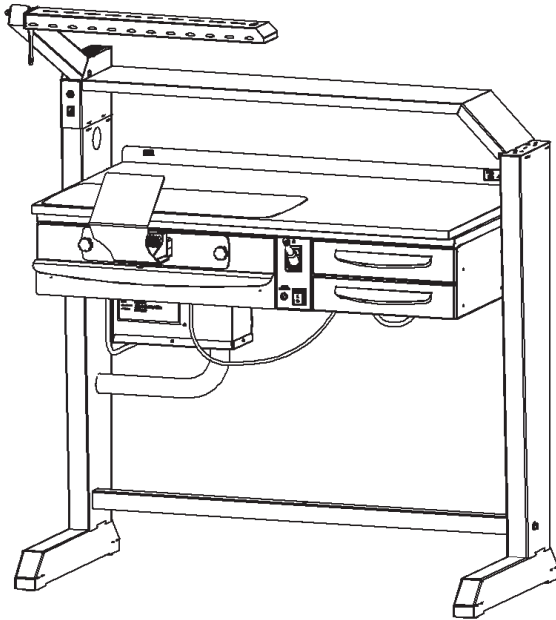
The products shown in this instruction booklet have been designed for use in dental laboratories and dental practices. Any other use must be regarded as incorrect and may be the cause of damage both to the equipment and the operator. In this case, the manufacturer does not accept any responsibility. The ideal environmental conditions for use of the equipment are:

- Indoor use
- Altitude of up to 2000 m;
- Temperature from 5 to 40°C;
- Maximum relative humidity of 80% up to 31°C in linear drop to 50% at 40°C;
- Supply voltage fluctuations +0 ±10% nominal voltage;
- Transient overvoltage according to installation category **II**, 2 degrees of contamination.

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TEKNO:

It is a slim and functional table, easy to set up and clean. It is a great solution for a modern dental technician and/or dental practice needs and ensures absolute freedom. There are four possible configurations and three different top heights (85, 89 and 93 cm).
All components are shipped in a single pallet.

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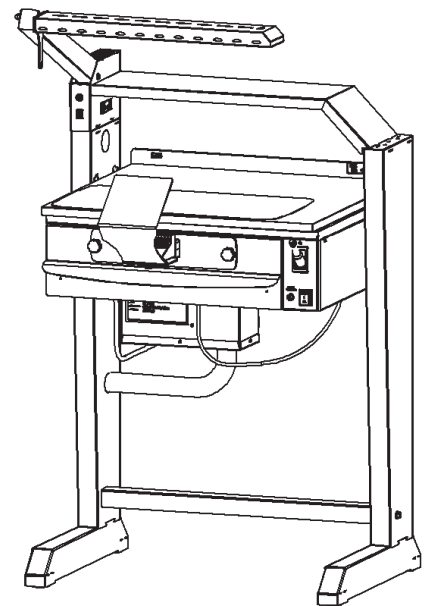
TEKNOmini:

It is a great solution for a modern dental practice needs and ensures absolute freedom. There are two possible configurations and three different top heights (85, 89 and 93 cm). All components are shipped in a single pallet.

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Various safety rules must be observed for the safety of the table:

- Position the elements on a flat stable surface;
- Check that the supply voltage, indicated on the table's label, corresponds to that of your mains;
- Assemble the table following carefully these instructions;
- Connect the table to a power socket with an efficient earth connection;
- After the installation, the connection plug must remain accessible to the user so that the machine can be disconnected during maintenance operations.;
- The lamp supply cable must be connected to the sockets inside the service raceway;
- Do not cover the machine's air evacuation outlets.

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• Attention! To maintain the protection level of the machine during maintenance operations, the operator must refer to the instruction manual.

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• Attention! Disregarding this warning may result in damage to equipment.

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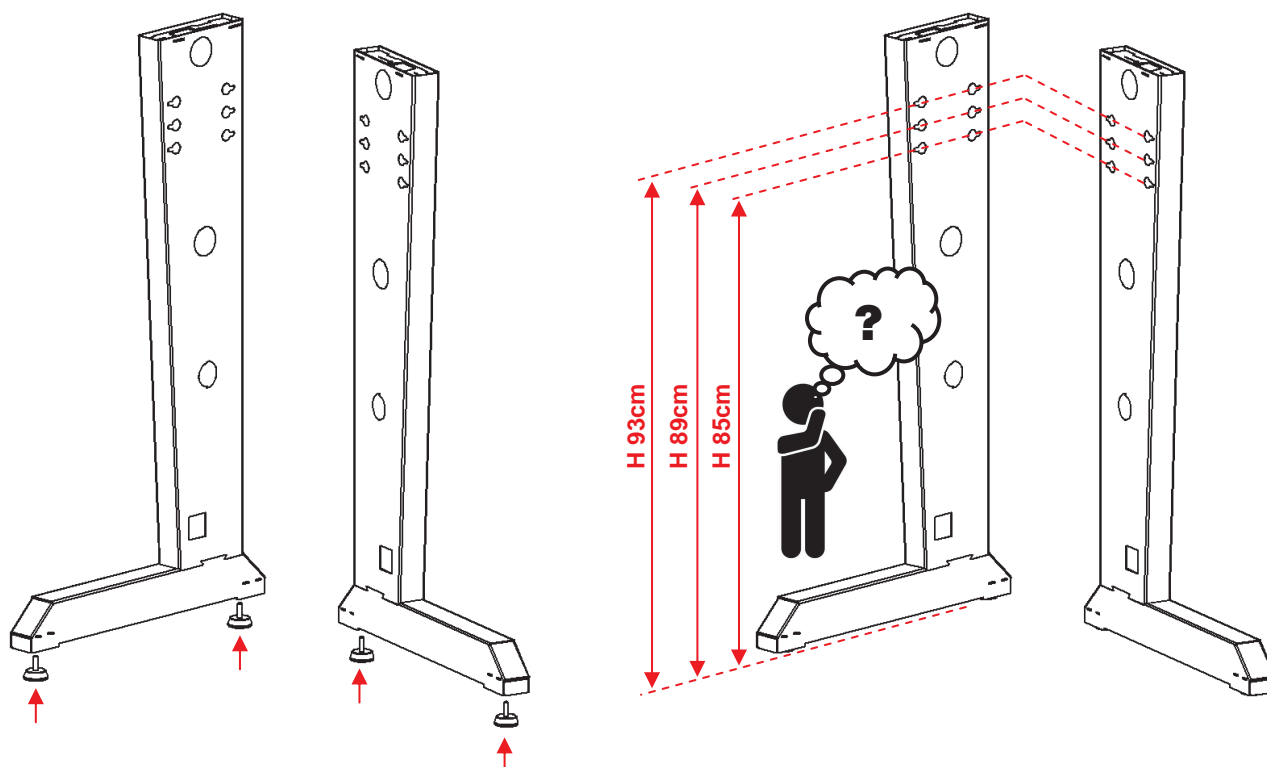
• Position the elements on a flat stable surface. Open the screws bag and take out the tools for the installation.

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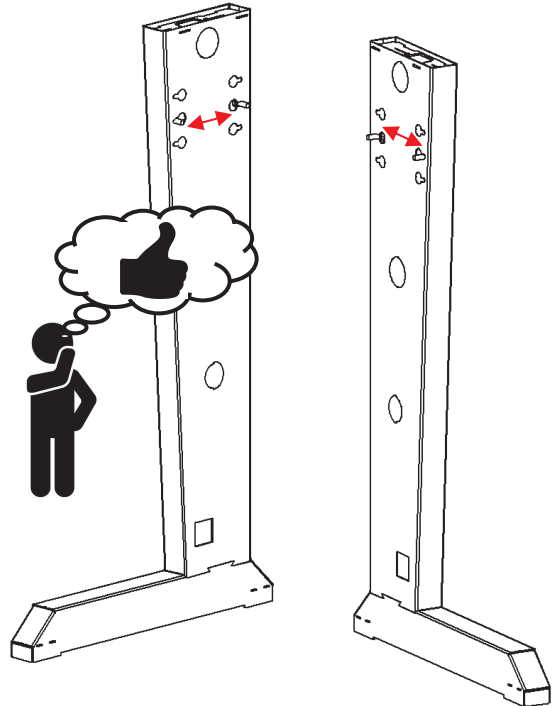
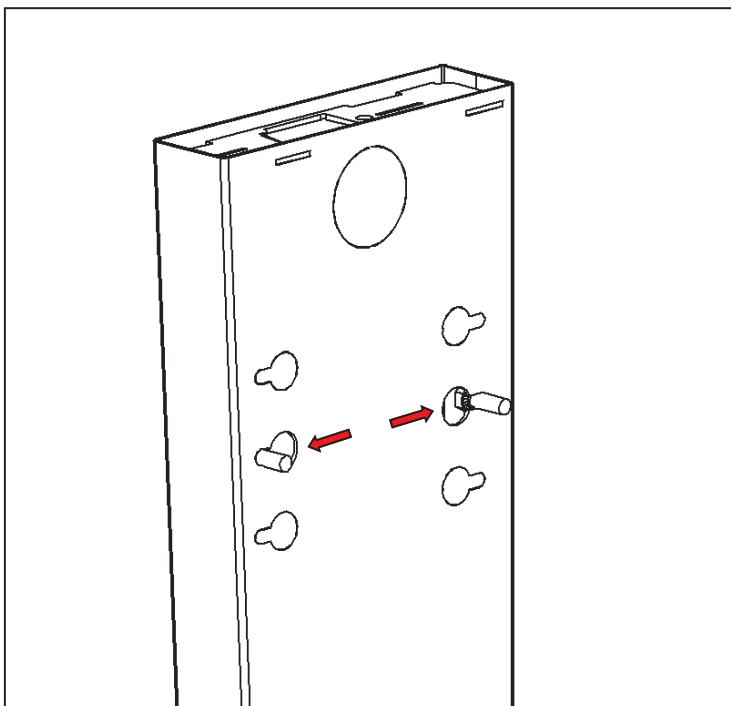
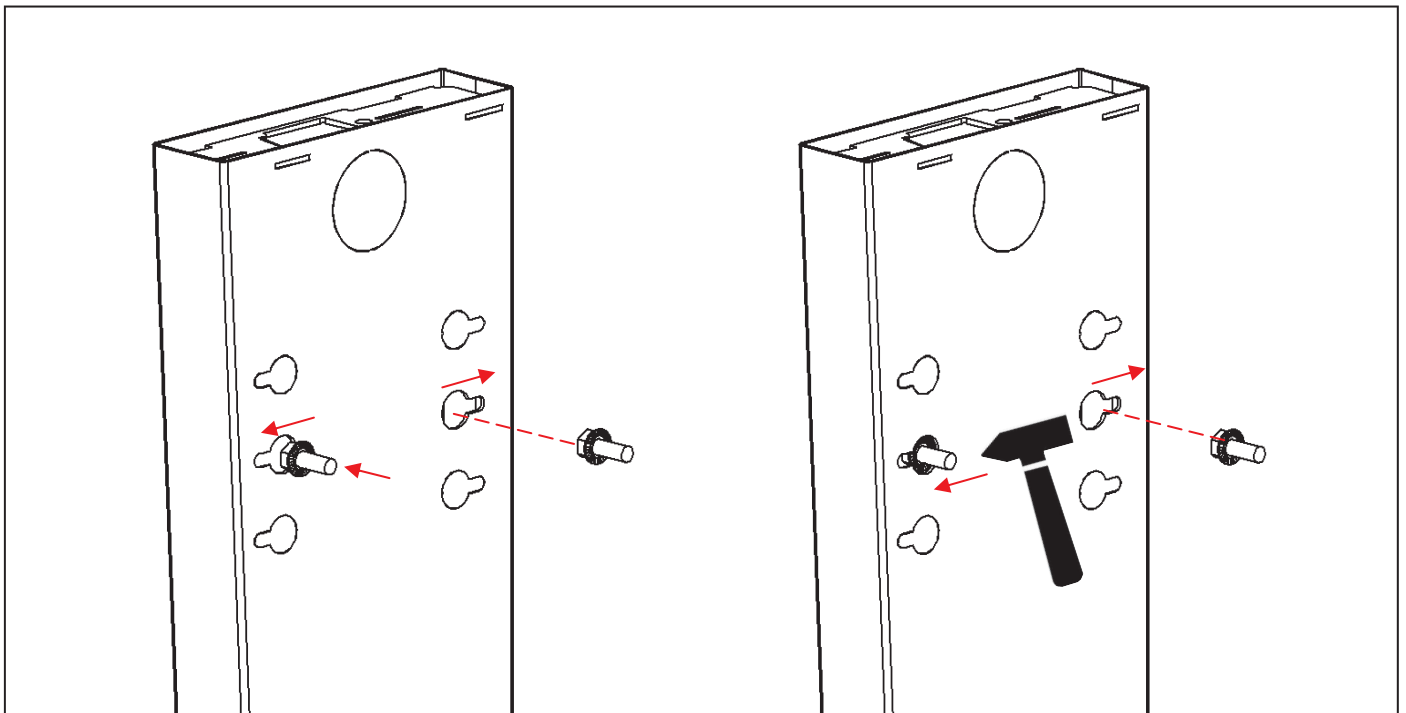
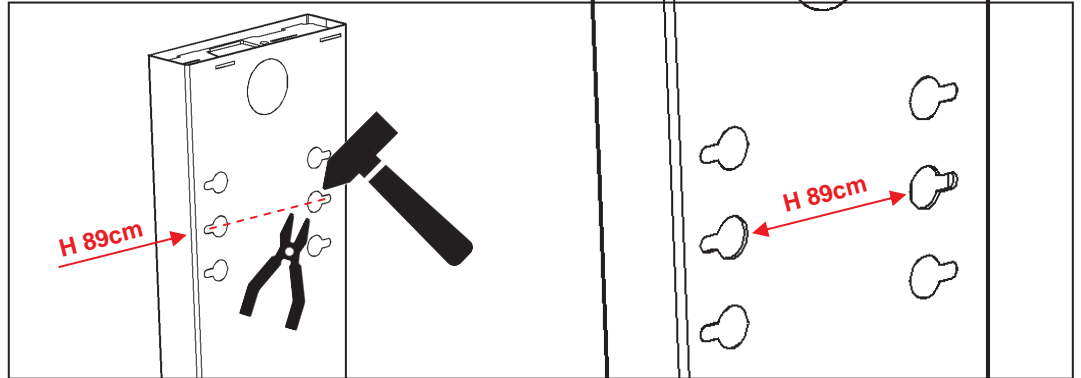
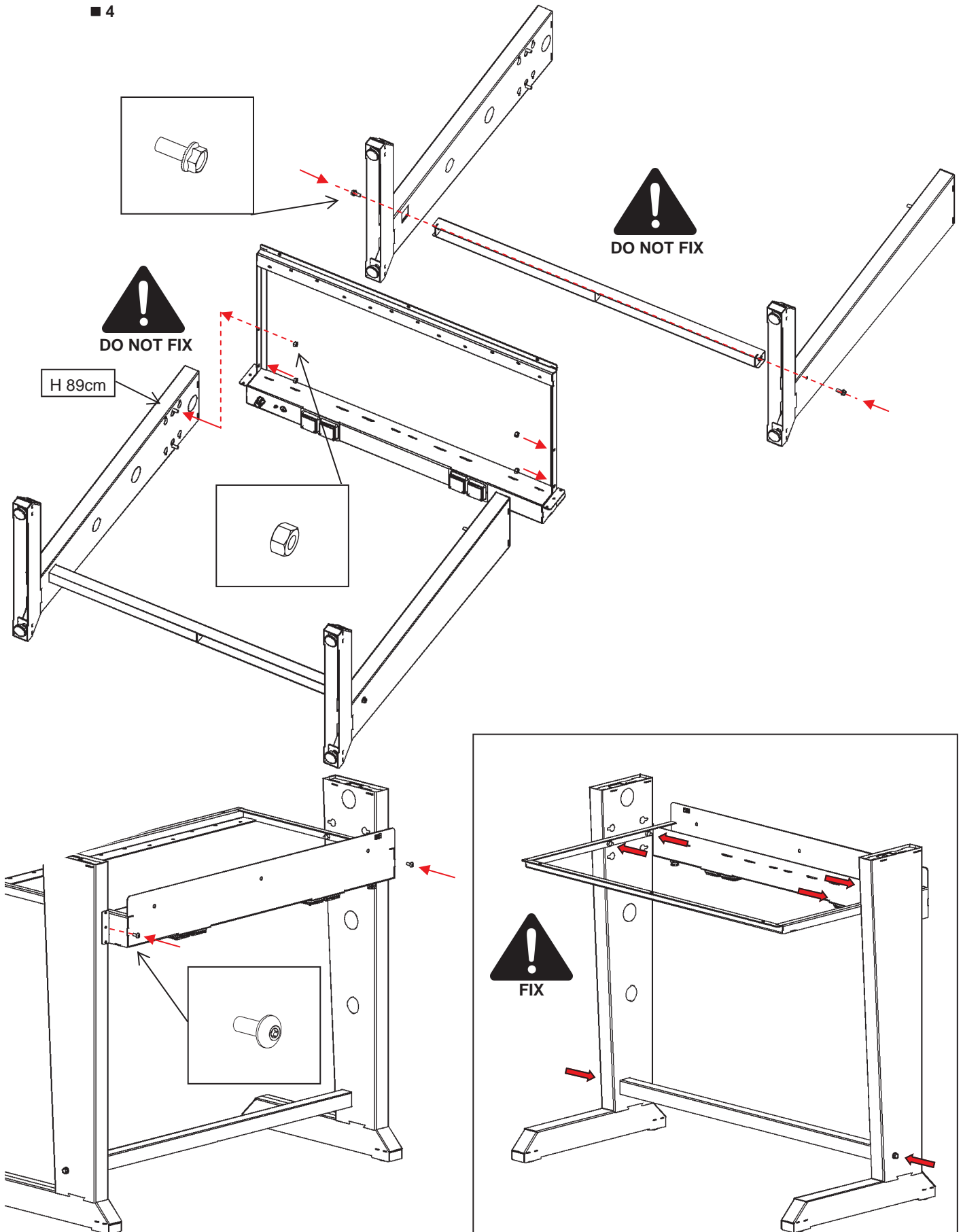
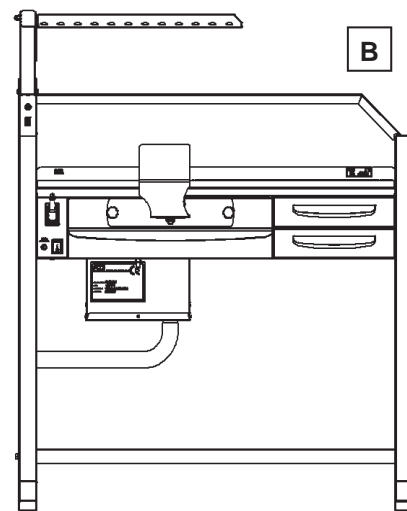
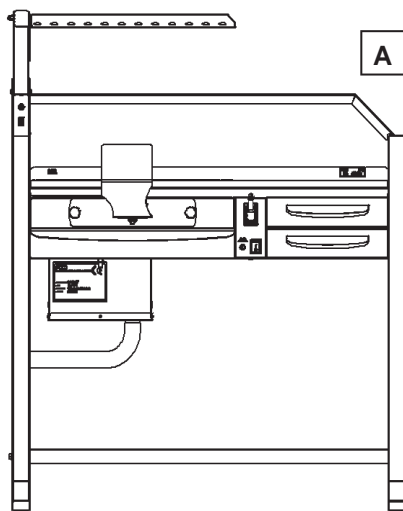
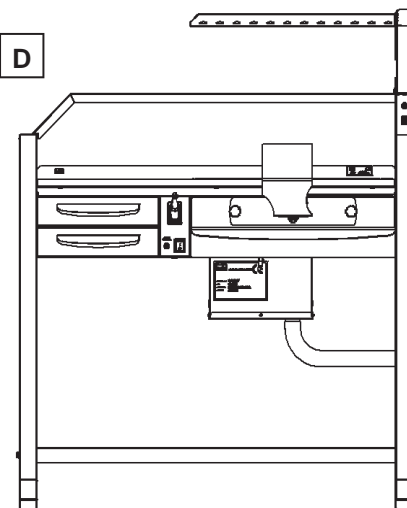
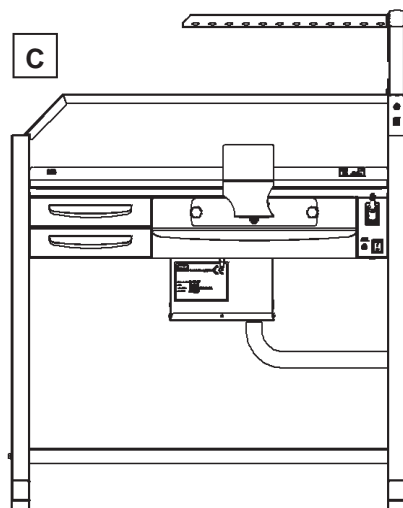
TEKNO table configuration:

- A. Table with left lamp
- B. Table with left lamp for left-handed
- C. Table with right lamp
- D. Table with right lamp for left-handed

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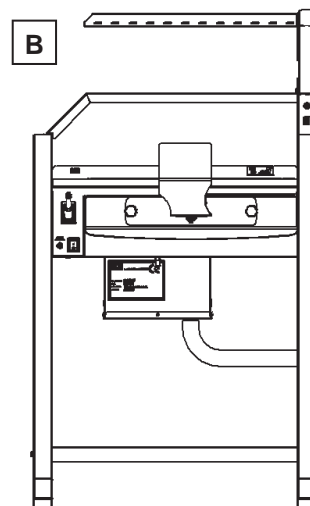
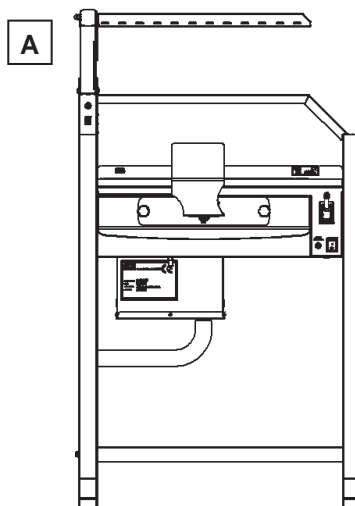
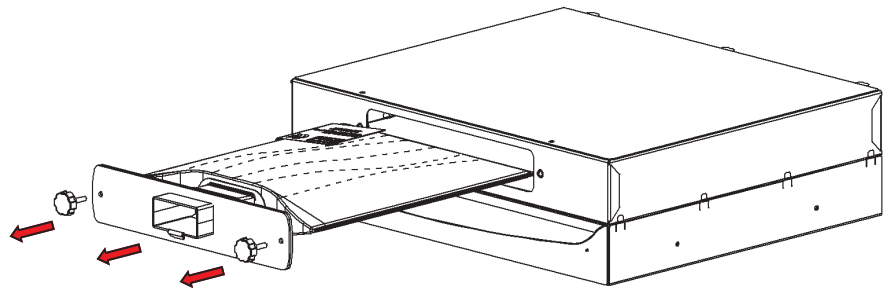
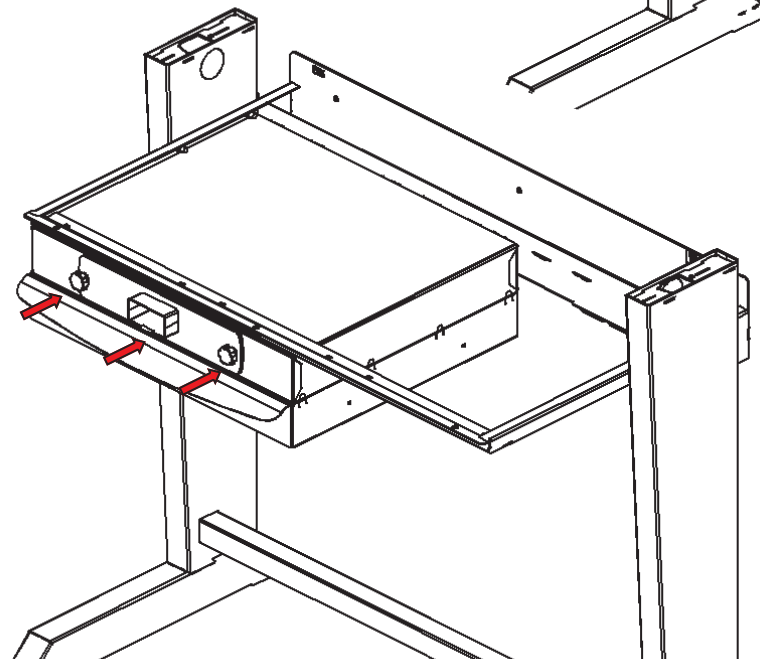
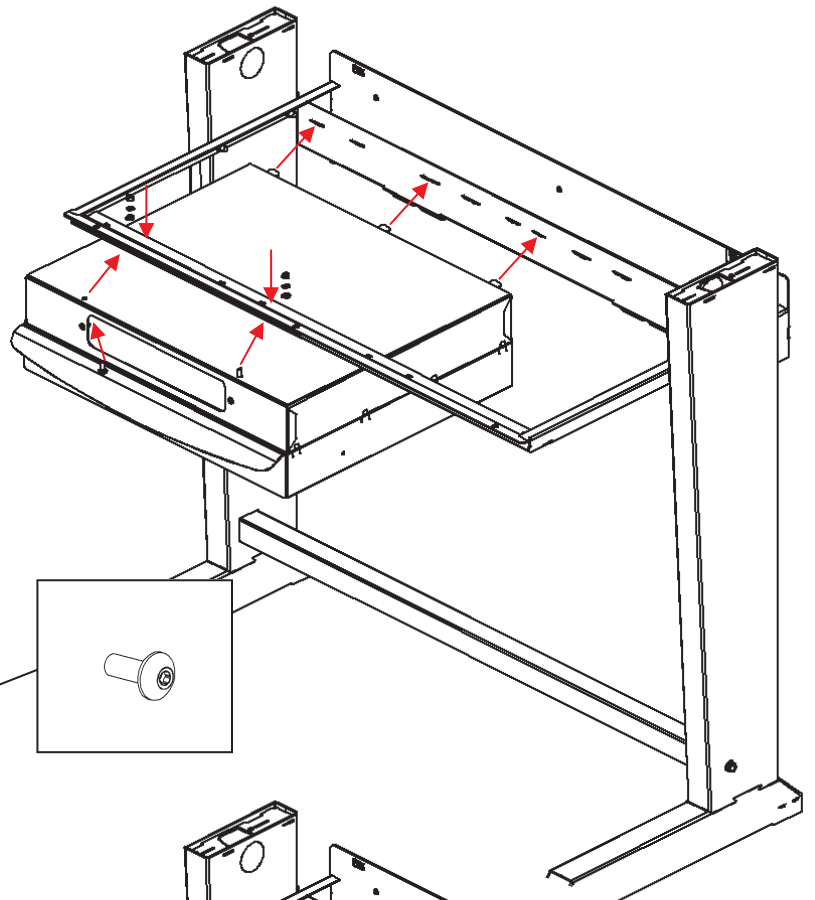
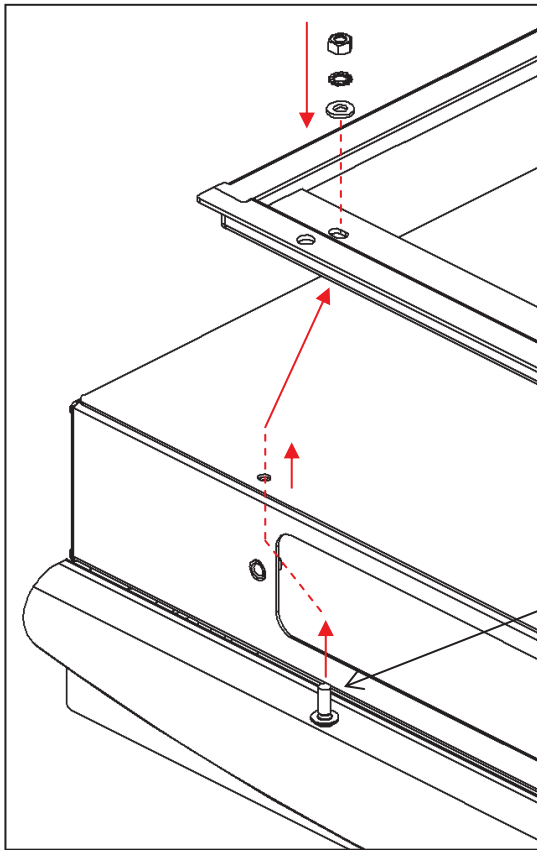
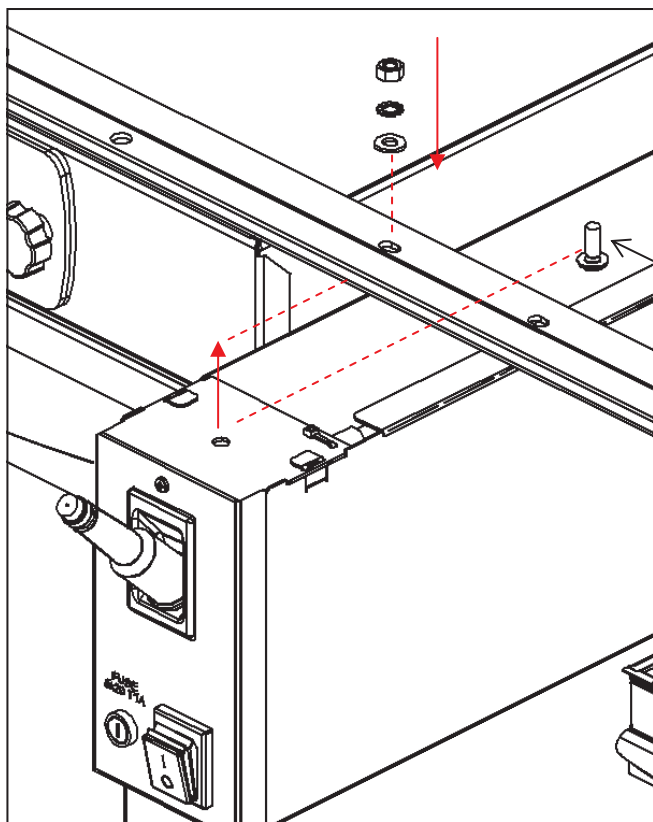
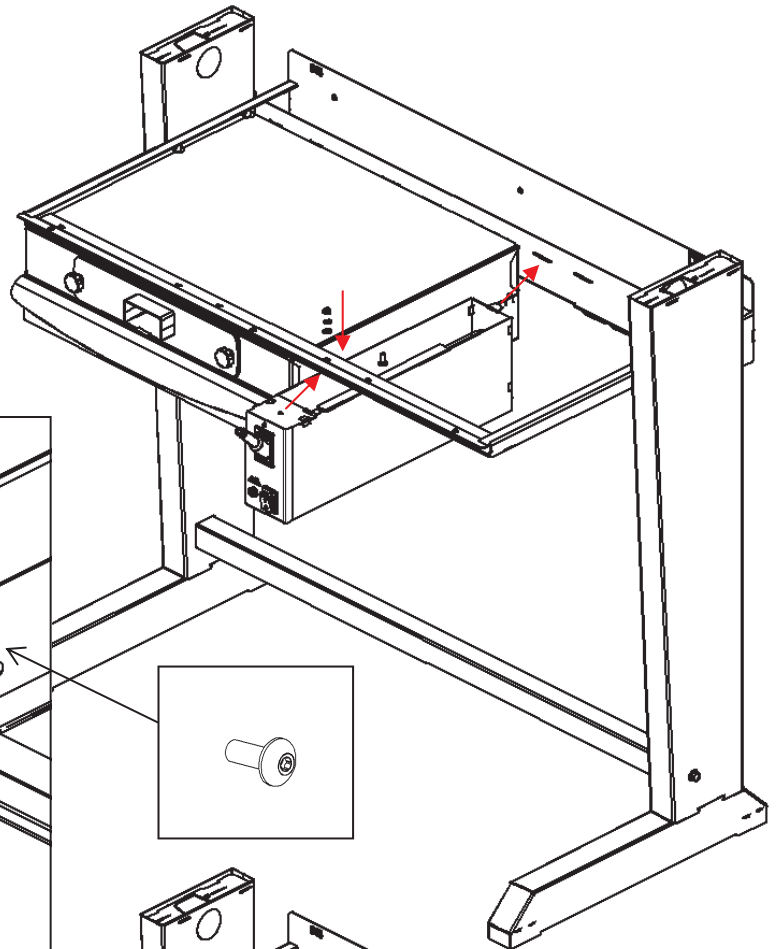
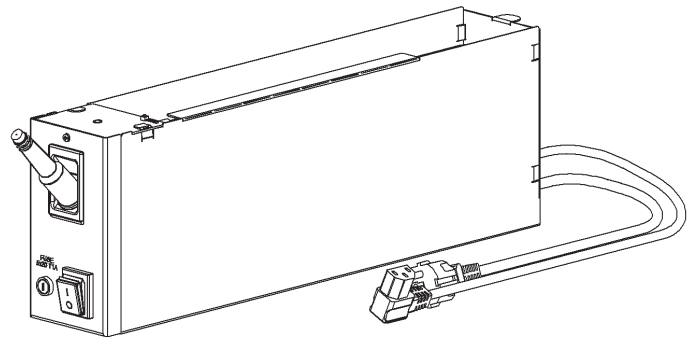
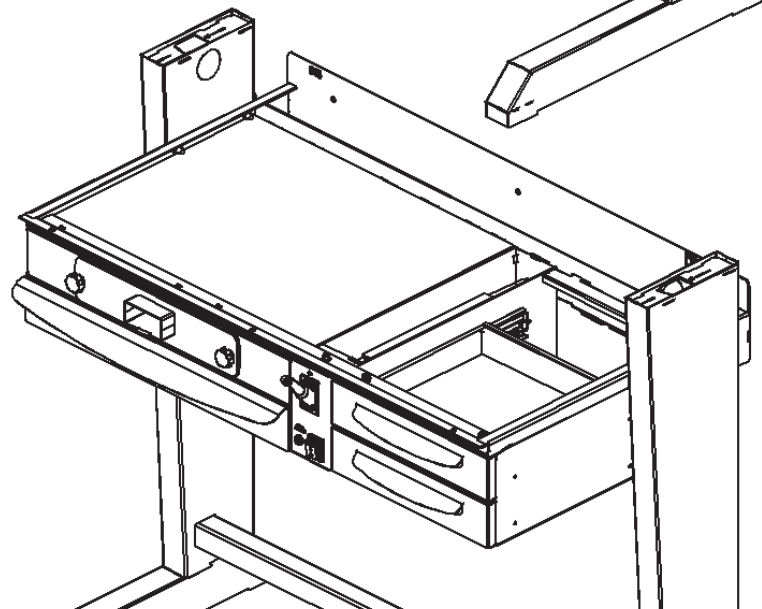
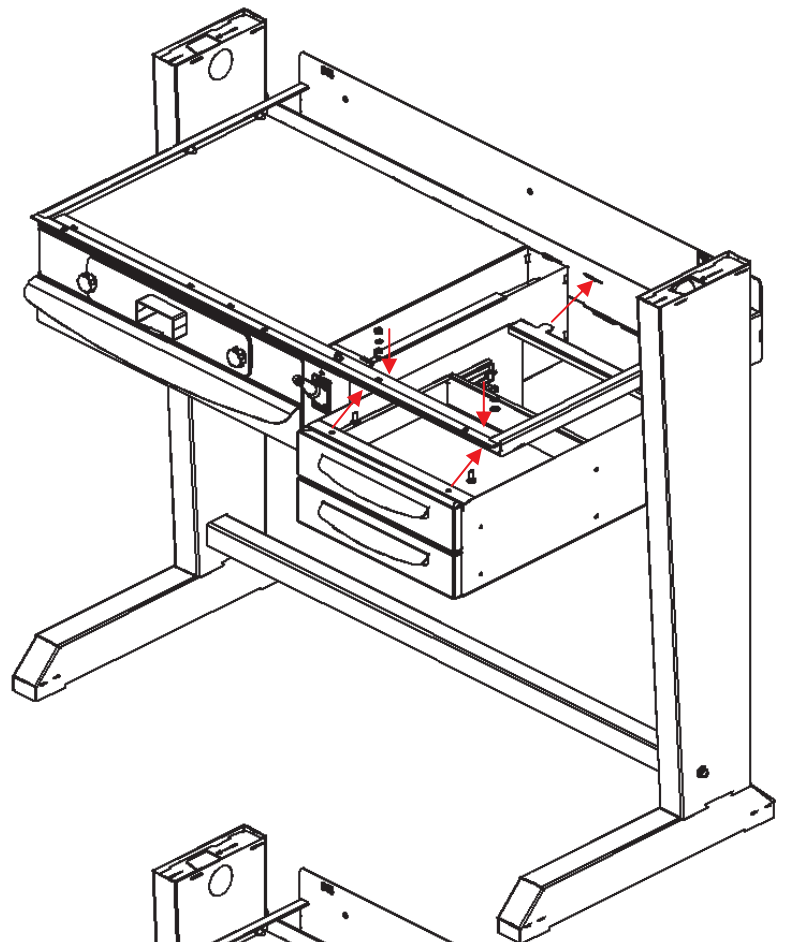
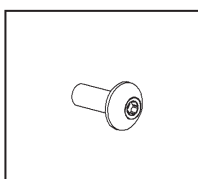
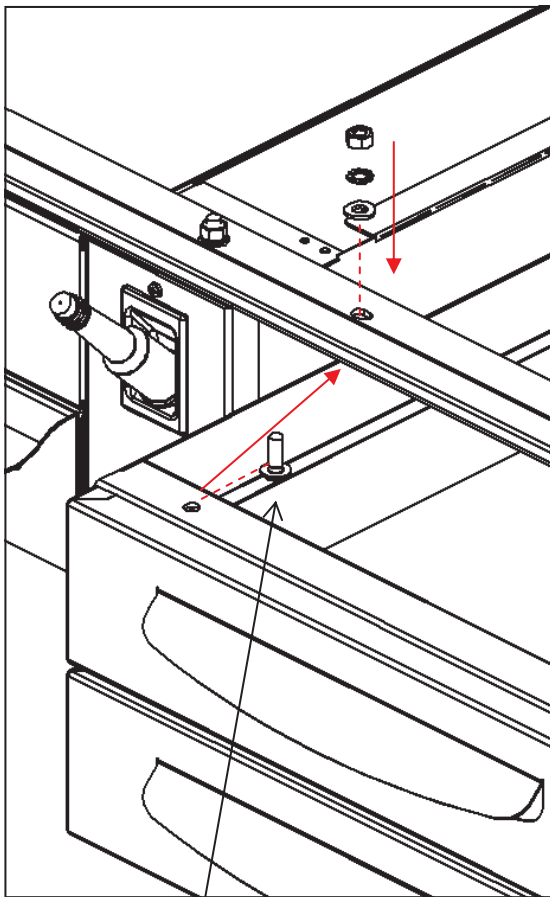
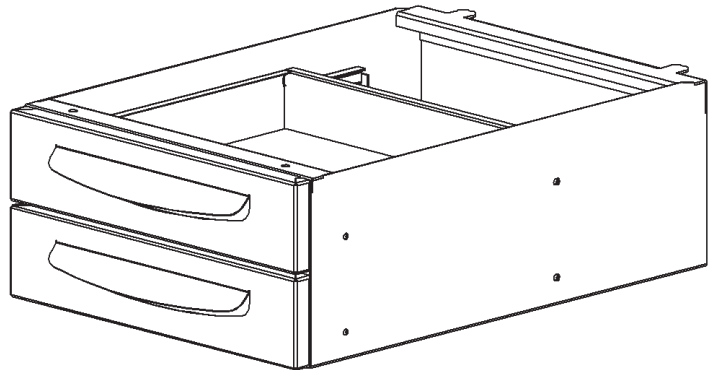
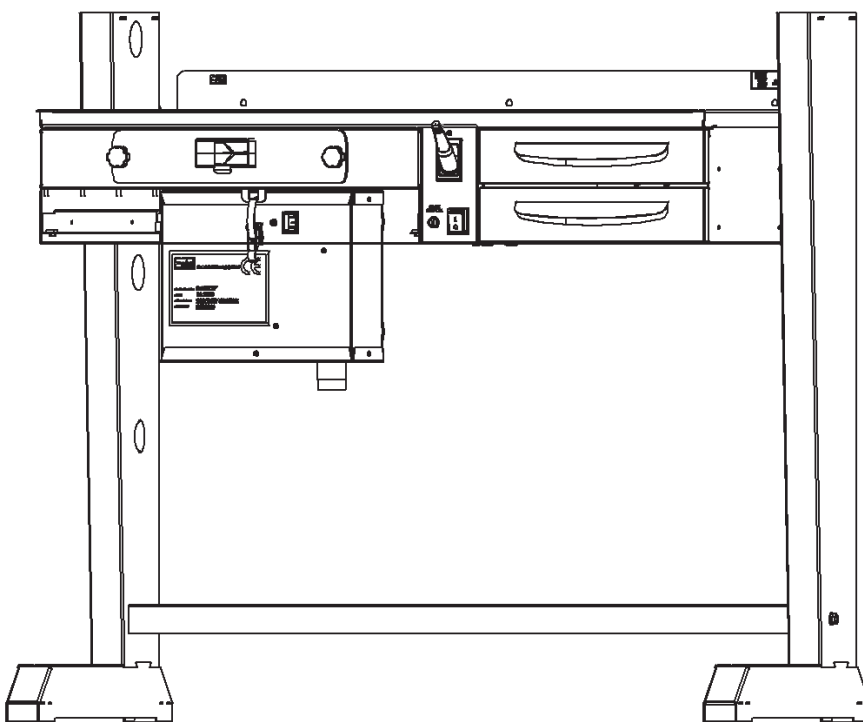
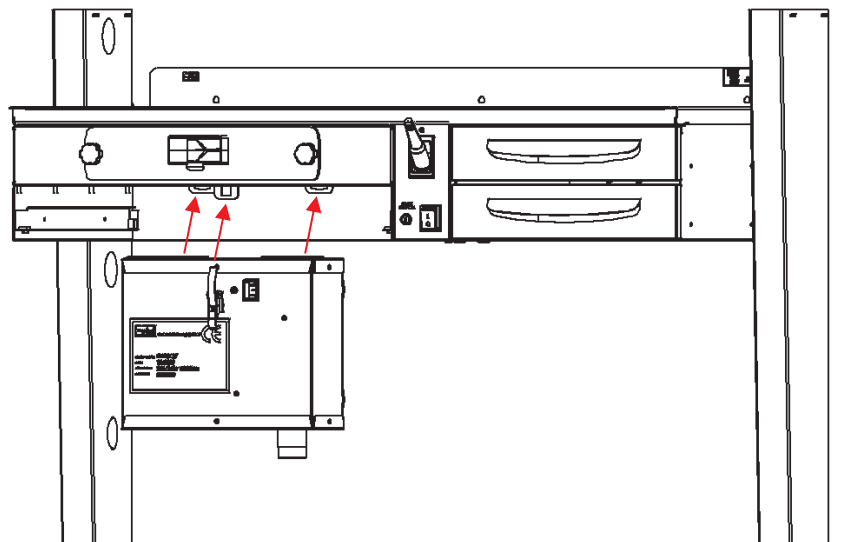
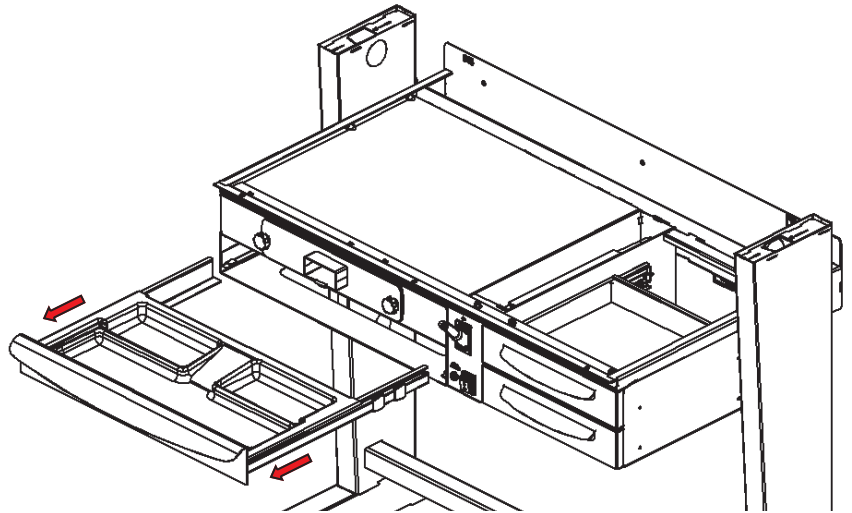
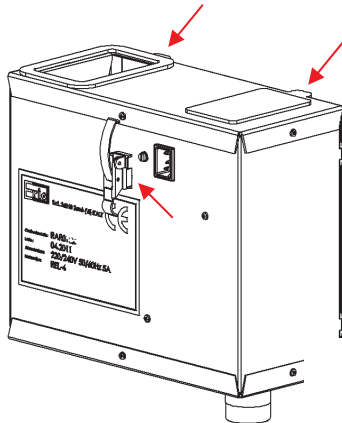
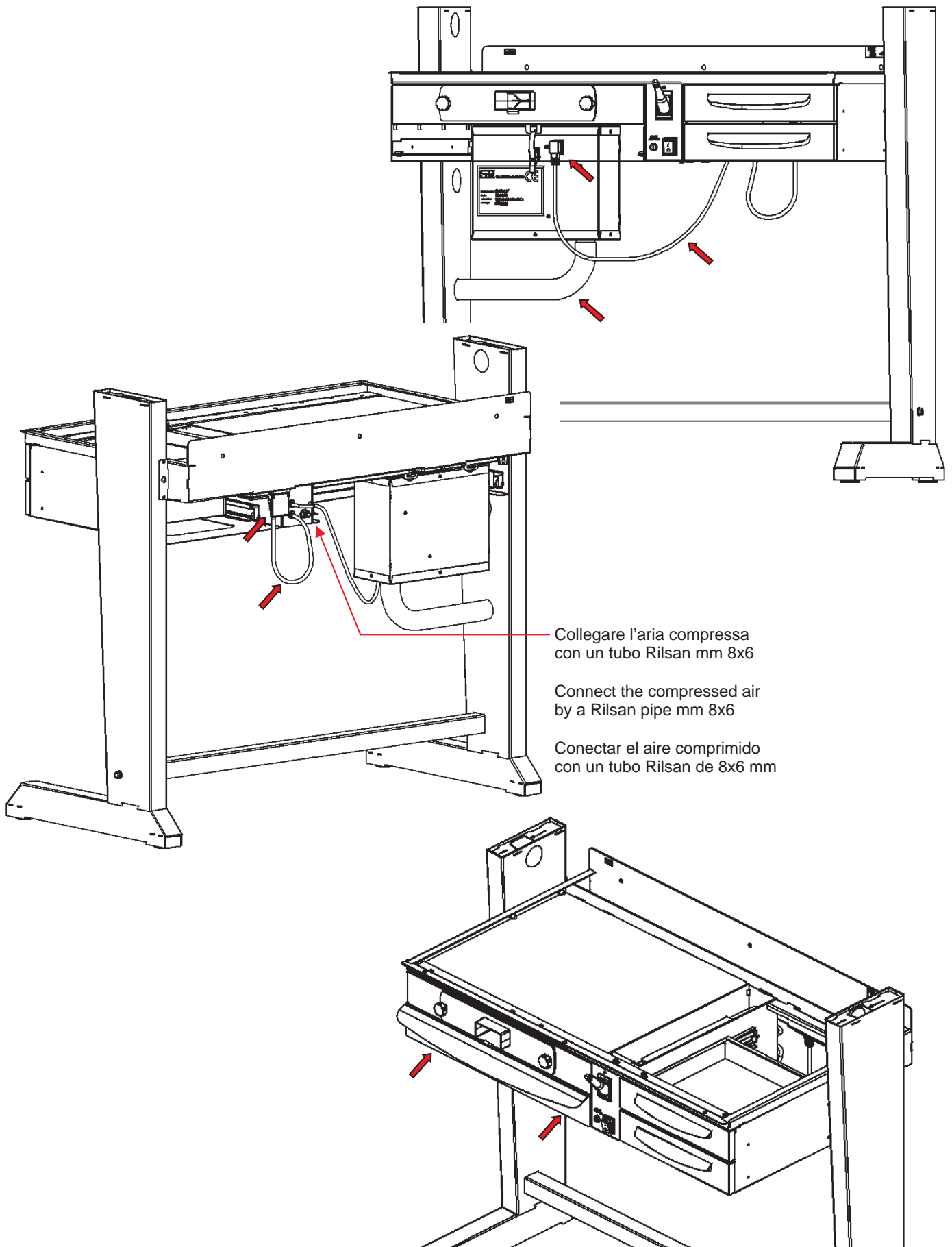
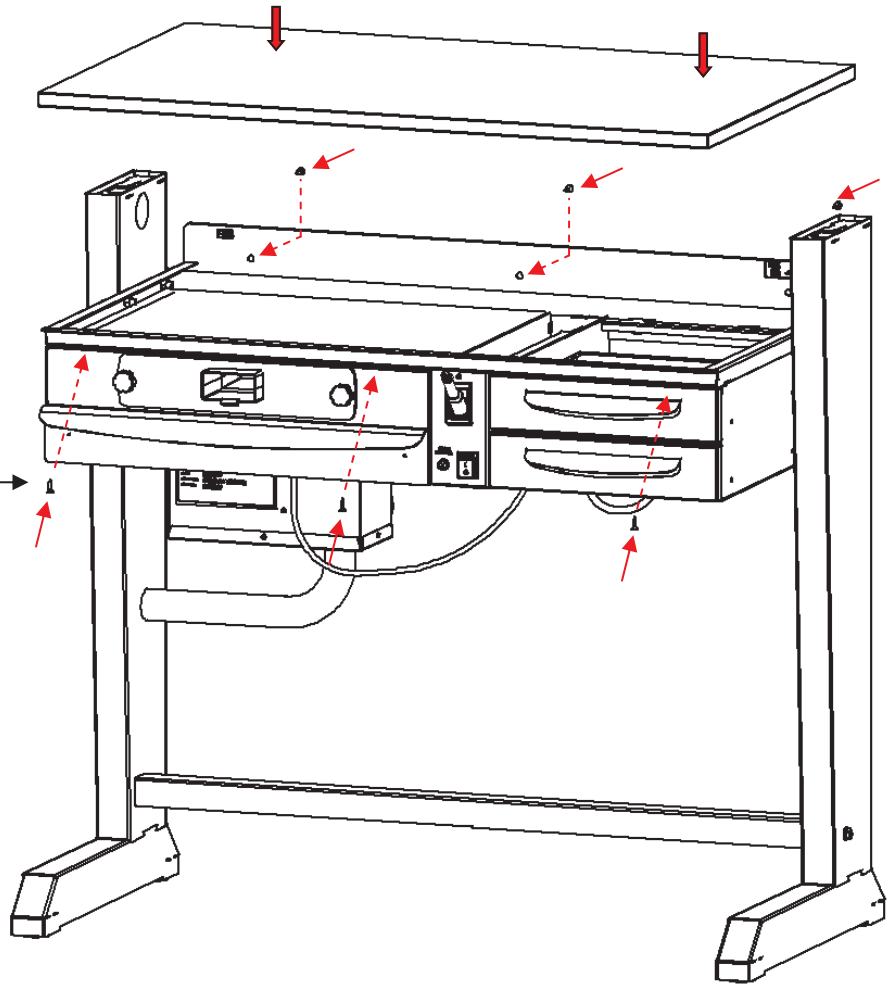
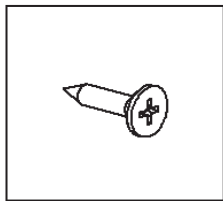
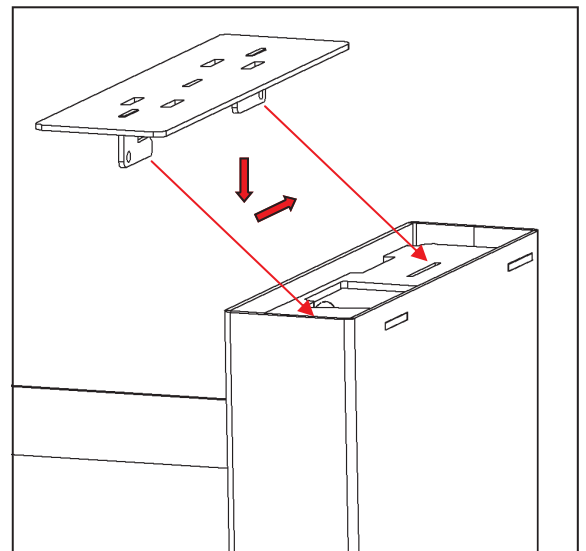
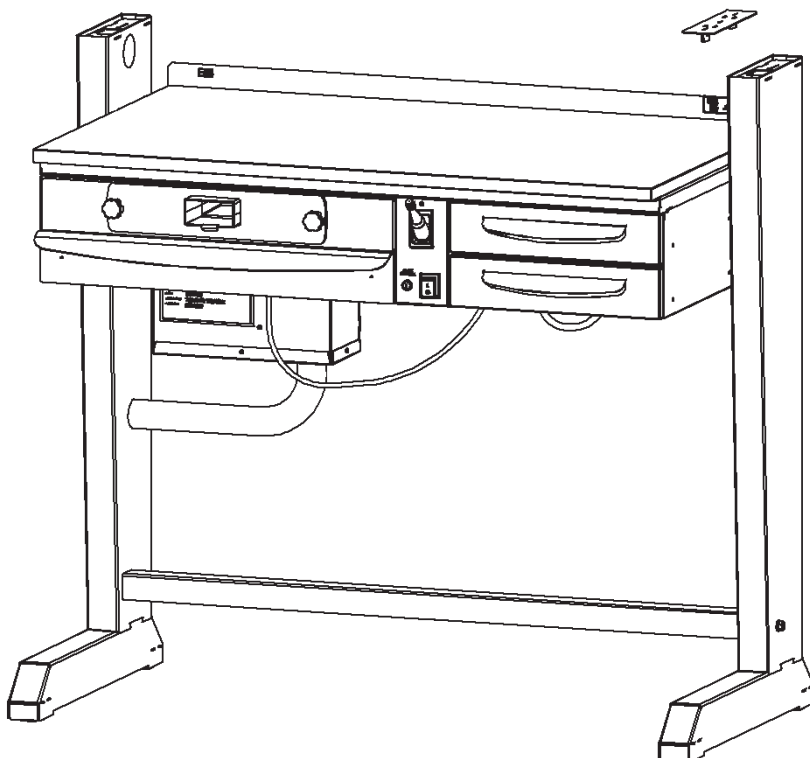
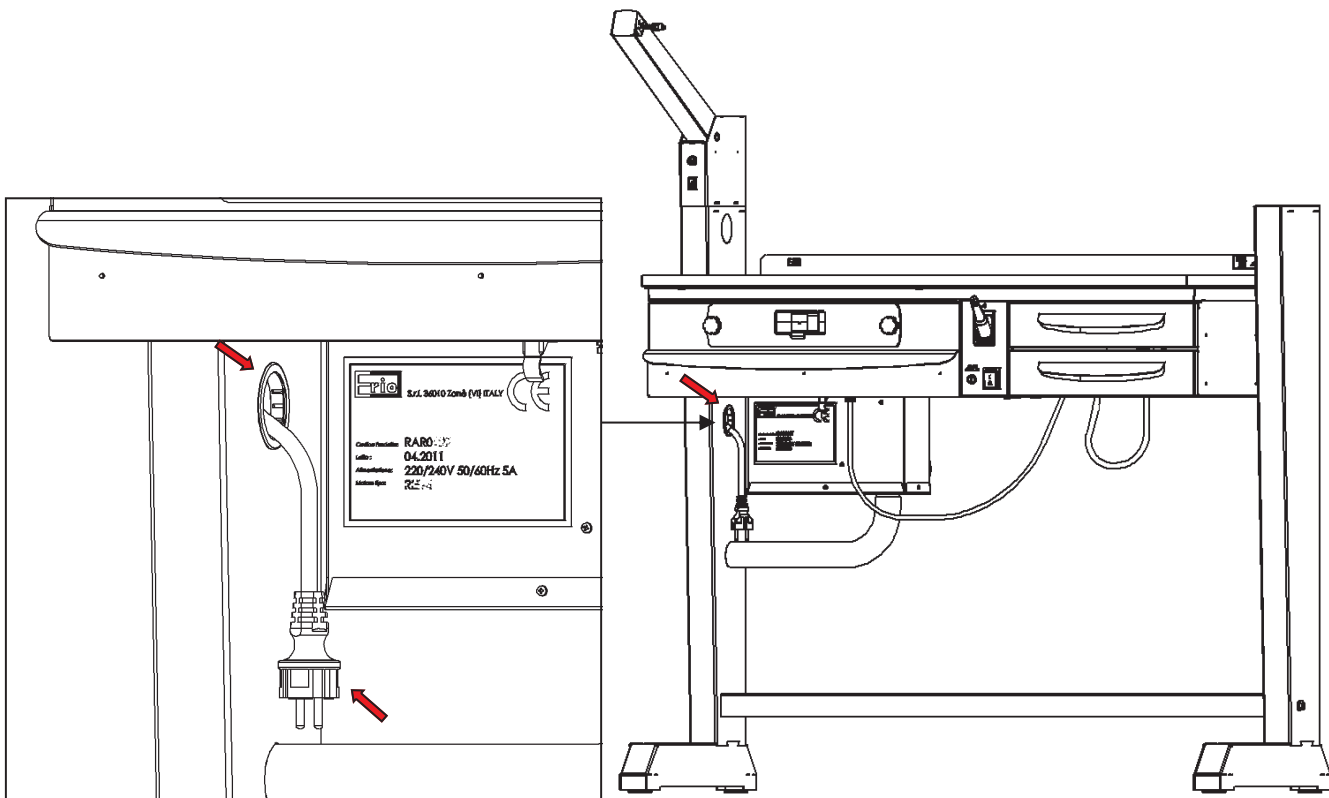
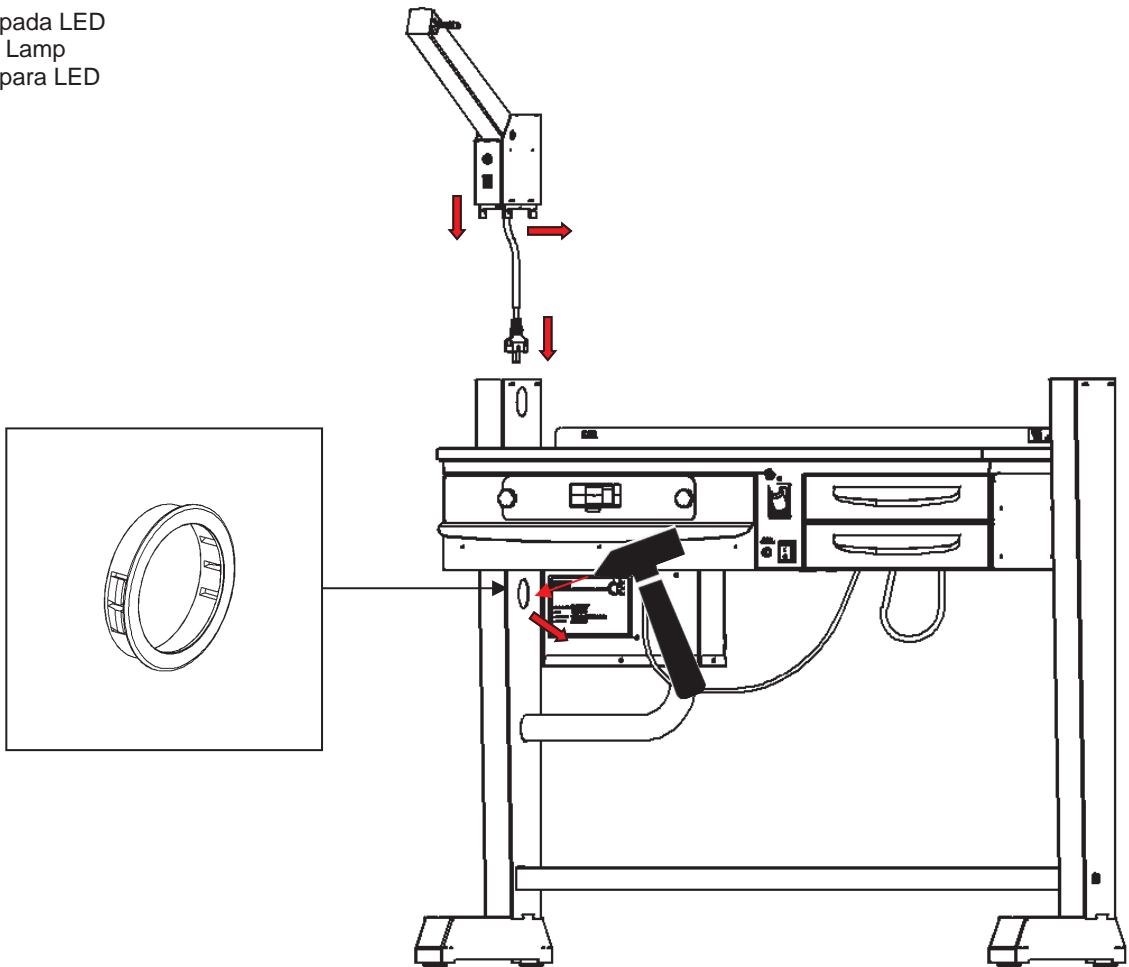
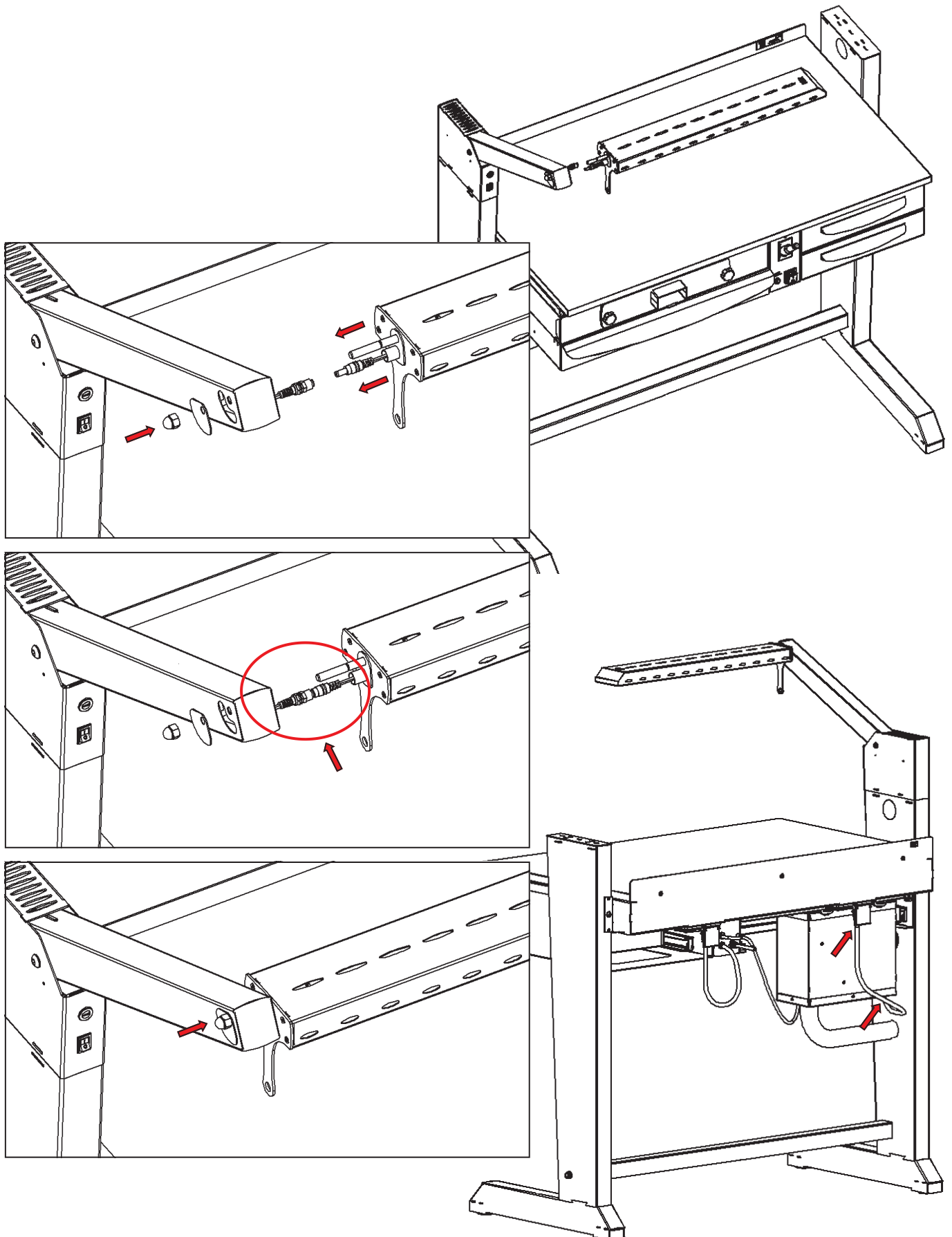
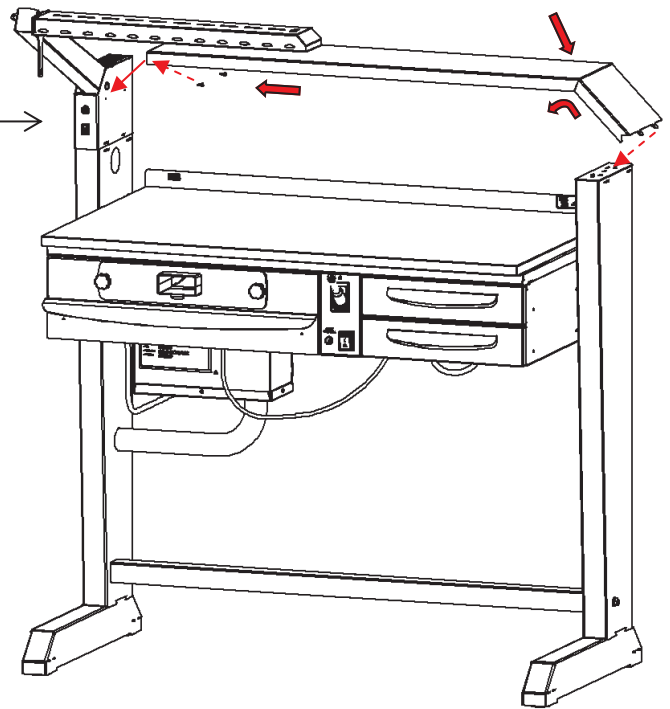
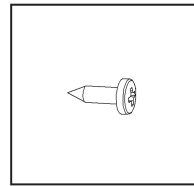
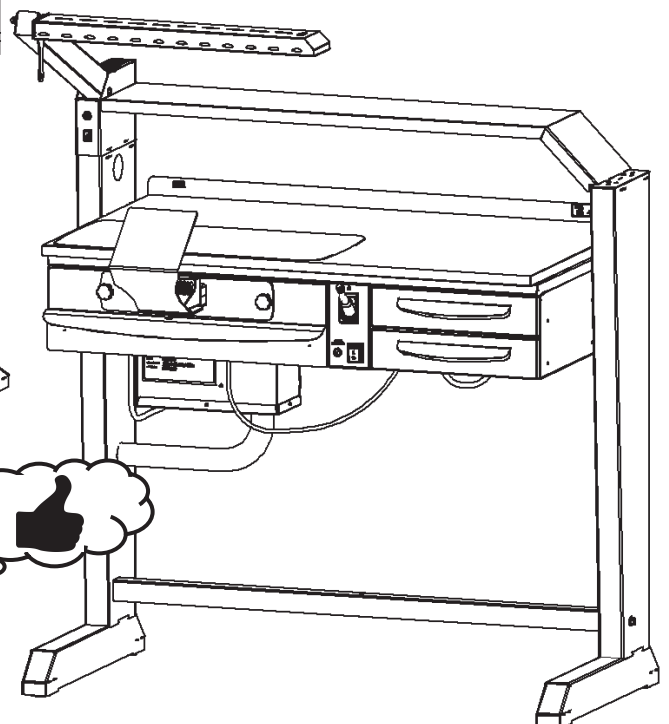
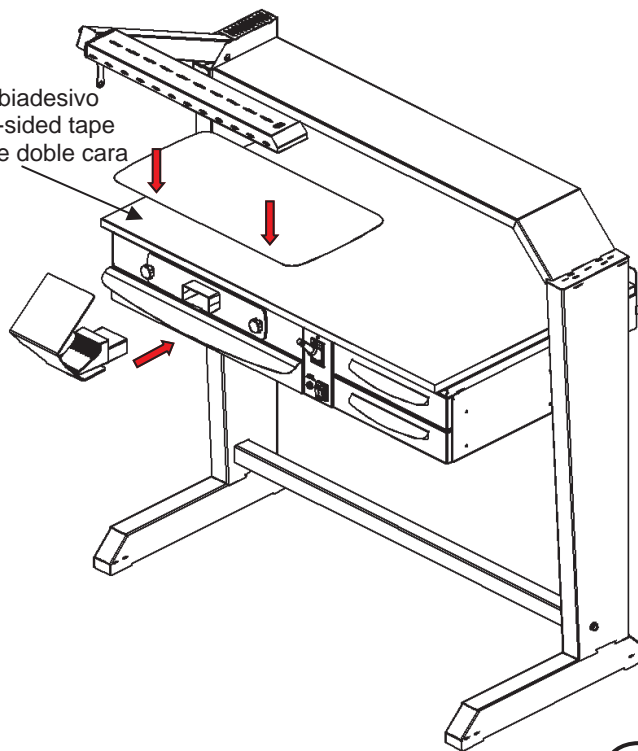
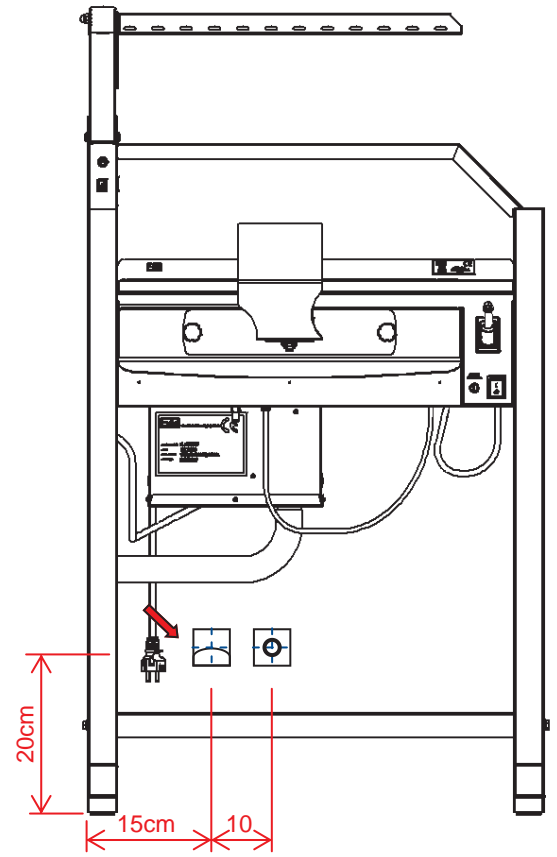
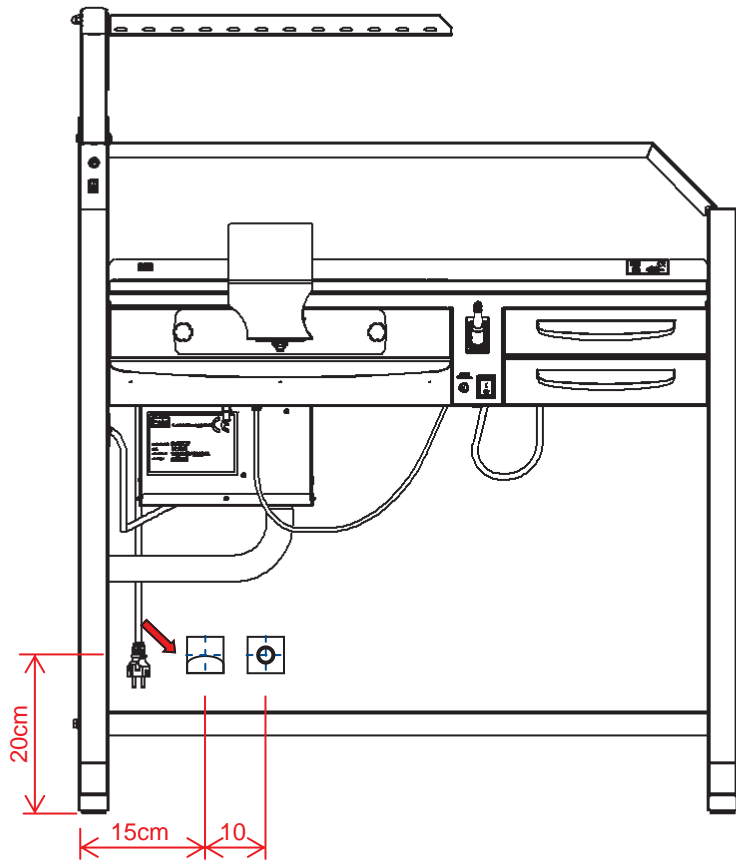
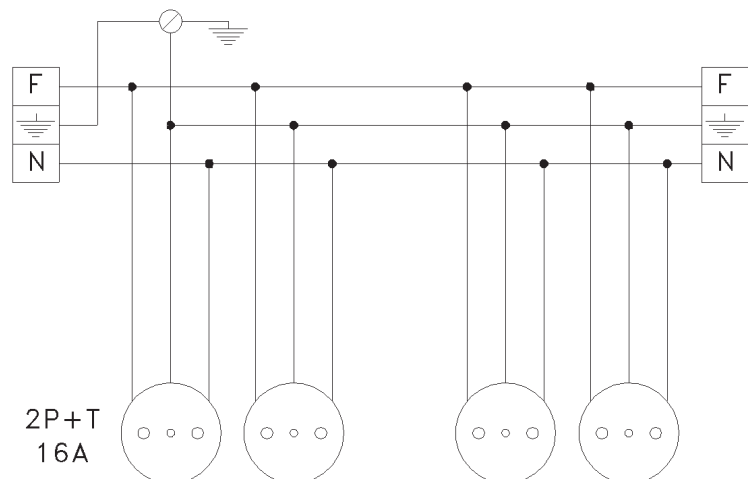
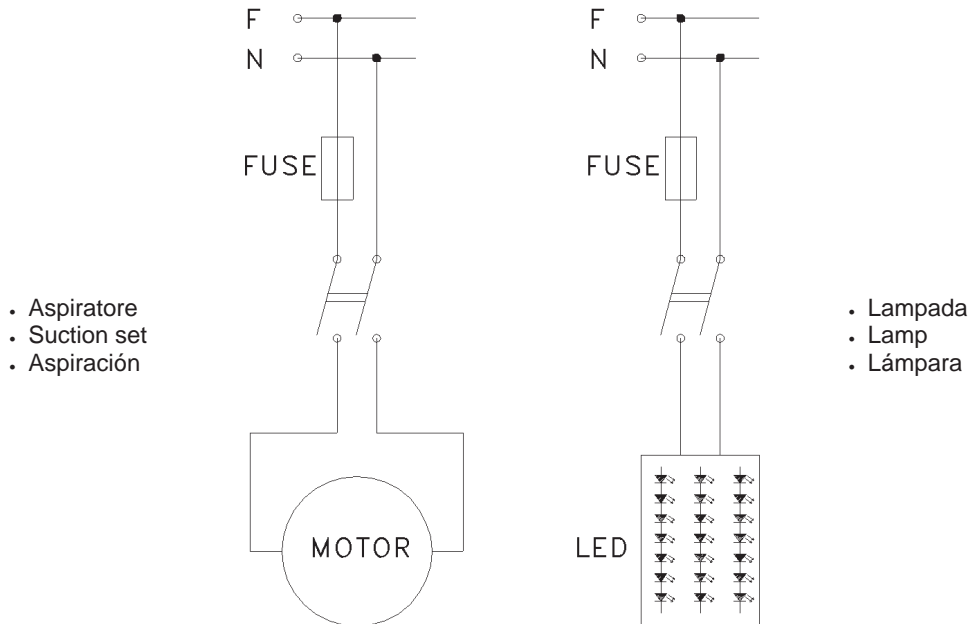
3.4 Ordinary maintenance of the machine

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- Replace the main filter bag when the suction becomes insufficient (4.1).

Observe the local waste regulation and accident prevention regulations when disposing of the product!

Wear the appropriate personal protection equipment according to the type of dust.

Note: Do not carry out maintenance operations other than those specified in the manual.

Any operation not envisaged in the documentation can be hazardous.

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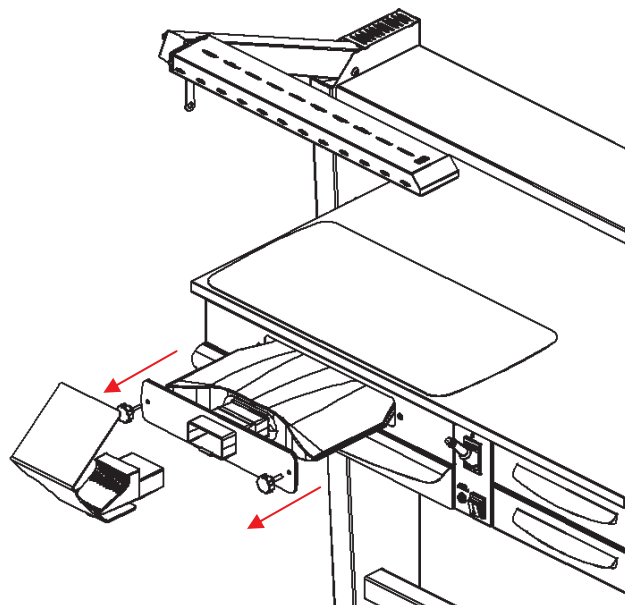
The filter maintenance operations can be carried out by the operator who uses the machine habitually provided that he/she is trained appropriately.

To open the filter panel unscrew the locking knob and pull it. The filter should be changed fairly regularly depending on the number of working hours of the extractor and, in any case, when the pores of the filter fabric are choked.

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
The suction motor maintenance and cleaning and/or repair operations must be carried out by qualified personnel authorized for the purpose after disconnecting the machine power supply.

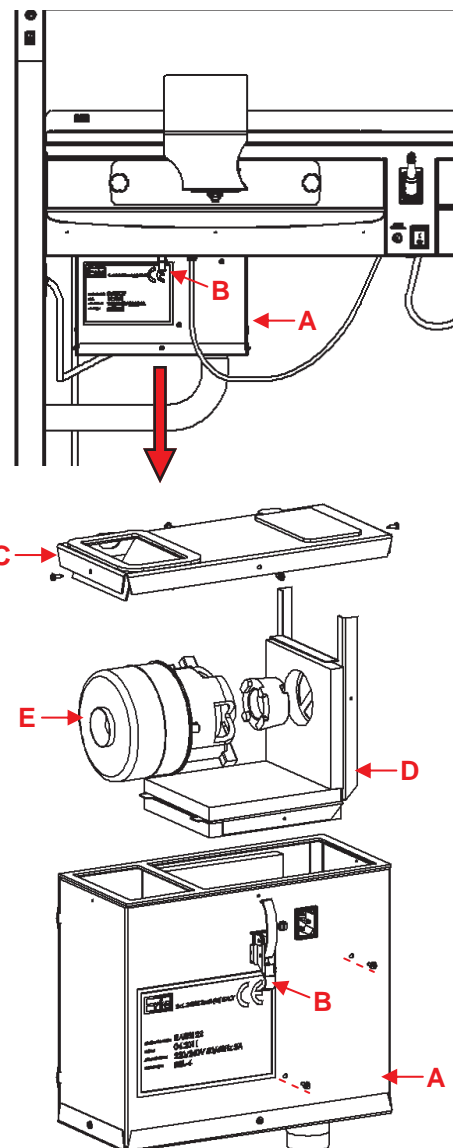Disconnect the electrical cable which powers the motor inside the box (A) and disconnect the discharge pipe. By actuating the lever (B) remove the motor box from the main basin and remove the cover (C). Disconnect the four screws that fix the motor plate support (D). After disconnecting the electrical socket contacts (after taking note of the exact position/colour of the wires) remove the motor (E) and replace it. Repositioning the motor, take a great of care that it adheres perfectly to the seal, if it does not there will be no suction. To complete the maintenance work, close the motor housing and carry out the disassembly instructions in the reverse order.

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The replacement of the fuse can be carried out by the operator who uses the machine habitually on condition that he/she is trained appropriately.

Unscrew the head of fuse carrier, remove and replace the fuse; close the fuse holder.

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We recommend daily cleaning to remove machining residues and prevent large deposits of material.

Use a moist cloth to clean the outside of the product; do not use alcohol or other solvents.

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The table is packed in a single carton placed on a pallet. Inside you will find all table components such as:

- Two support sides
- Supporting cross
- Foot rest
- Suction set
- Control board
- Drawers unit (only in the **TEKNO**)
- Motor box
- Top
- Upper closing plate for the leg
- Lamp
- Working shelf
- Stainless steel top protection
- Vented peg with plastic screen
- Screws bag
- Instructions

Inside of the screws bag there are some tools that you need to assembly the table.

The loading and unloading operations should be carried out using a transpallet and for the placement of mounting components two people are necessary.

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In its various versions conforms entirely to the following standards and directives: **Dir. 2004/108/CE, Dir. 2006/95/CE**

Erio srl. also declares that the above mentioned machines maintain their compliance with the standard if: they are installed, used and regularly maintained and cleaned according to the recommendations supplied in the instructions for use; no component has been added, removed, modified or replaced without the express authorisation of Erio srl.; no other electrical or pneumatic appliance not expressly authorised by Erio srl. has been connected to the machine after delivery.

Responsibility for damage caused by defective products (Dlgs 206/05 and Dir.1999/34/CE) expires 10 years after the date of sale, and then rests with the user.

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DATA: 10.02.2025 REV.2

PAGINA: 30/32

WARRANTY TERMS ERIO PRODUCTS
General 12-month quality warranty

All products are guaranteed for 12 months from the date of delivery against material and manufacturing defects. Products found to be defective during the warranty period will be repaired at the Erio works in Zanè (Vi) free of charge or replaced, but no work will be carried out at the Customer's premises.

The analysis of the defectiveness will be carried out by Erio S.r.l. at its works and, if the outcome is positive, Erio S.r.l., at its sole discretion, will replace the defective piece/component free of charge. Shipping and other expenses are not covered by this guarantee in any way.

Products must be returned intact in their original or similar packing and with appropriate outer packaging in order that there will be no damage suffering during transport. Any parts damaged during transport because of poor packaging or other causes not attributable to Erio, and highlighted on arrival of the returned product, shall not be credited in any way.

Please include all your data in the pack along with the reason for the return, the number and date of the document with which you received the material. The procedure for returning products must be agreed on in advance by e-mail.

Warranty on painted parts:

Erio dental laboratory equipment is built in oven-epoxied galvanised steel plate. The duration of the warranty against corrosion of these materials is 5 (five) years.

The warranty on the painted is forfeited if pickling acids, strong bases or acids and/or corrosive materials are used on them.

Warranty on stainless steel parts:

The bench tops in AISI 304 stainless steel are built with certified materials and the guarantee against corrosion is for 5 years (five).

The warranty is forfeited when:

- A) use is made on the stainless steel parts of materials not permitted according to the merchandise tables attached to the tops.
- B) there are phenomena of electrolysis and electro-corrosion caused by special equipment or failure to earth the stainless steel tops.
- C) episodes of "pollution" occur with iron particles (filings) deposited on the stainless steel surfaces.

Warranty on the electric and electronic parts:

The electric and electronic equipment is guaranteed against manufacturing defects for 1 (one) year. The warranty expires when repairs or tampering are carried out by personnel not authorised by Erio srl in Italy, or by an Official Erio Dealer in other territories.

The warranty for electric motors is valid for up to 2500 hours of effective use or for one year, whichever is the more severe.

Starting date:

The warranty on Erio s.r.l. products runs from the date of installation of the material in the user-customer's laboratory. This date must be witnessed by the waybill which accompanied the goods or by the date of the test report issued by the personnel responsible for assembly.

Exclusions:

It is understood that the guarantee expires at the moment the products supplied by Erio S.r.l. are used outside the field of application for which they were designed.

Damage caused by improper use of the products is not covered by the warranty, and no responsibility is accepted for damage due to such improper use.

The warranty on electric motors of suction groups expire if you use non-Erio filters.

The guarantee is subordinate to correct installation.

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